## [EN] Product Description

Stratamed is a flexible, full contact, non-resorbable, occlusive wound dressing for damaged or compromised skin.

When used as directed, Stratamed dries to form a protective wound covering that is gas permeable and waterproof which hydrates and protects compromised skin areas and superficial wounds.

Stratamed can be applied to superficial wounds and compromised skin surfaces including those following laser treatments, chemical peels, dermal abrasion, surgical incision and excision, and 1<sup>st</sup> and 2<sup>nd</sup> degree burns.

### How Stratamed works

Stratamed dries to form a thin, flexible, occlusive wound dressing which hydrates, covers and protects affected areas.

Stratamed helps to promote a moist wound healing environment for superficial wounds and burns.

Moist wound environments promote faster reepithelialization\*, relieve itching, burning and discomfort, reduce redness and swelling associated with insulated skin.

Stratamed is a self-drying, non-sticky, transparent gel formulation that lightly bonds to the most superficial damaged skin layer.

### Intended use

Stratamed is an occlusive wound dressing intended to cover a wound, to provide and support a moist wound environment, and to allow the exchange of gases such as oxygen and water vapor through the device.

Because Stratamed is a gel, it is easily applied to large exposed surface areas and contoured skin, like the face, neck and chest. Stratamed can be used with or without a secondary protective dressing.

Stratamed creates a moist wound environment.

**Stratamed was developed for use on all types of superficial wounds, burns, skin tears and abrasions including those resulting from:**

- Laser therapy (ablative and nonablative), including tattoo removal
- Chemical peels
- Photodynamic therapy (PDT)
- Dermal abrasion and needling
- General and cosmetic surgery
- Burns (1<sup>st</sup> and 2<sup>nd</sup> degree)
- Trauma
- Bites
- Grazes and abrasions
- Cryosurgery
- Acne and other diseases of the skin
- Radiation therapy

Stratamed may be applied to superficial wounds, burns and areas where skin integrity has been compromised.

Stratamed may also be used in conjunction with other adjunctive treatments to improve overall results.

Stratamed contains no alcohols, parabens or fragrances and is transparent and odorless.

Stratamed is suitable for people with sensitive skin.

### Directions for use

Ensure that the affected skin or wound is clean.

Pat dry as much excess exudate or wound fluid from the wound as possible prior to gel application.

For best results Stratamed should be maintained in continuous contact with the skin (24 hours a day/7 days a week). If Stratamed has been removed by washing it should be reapplied.

When applied correctly to exposed areas, Stratamed should be dry in 5–6 minutes.

If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

**On damaged or broken skin not requiring a secondary dressing:**

Apply a very thin layer of Stratamed to the affected area and allow the gel to dry.

Stratamed should be applied once or twice daily or as advised by your physician to maintain continuous contact.

Once dry, Stratamed may be covered by sunscreen, cosmetics and/or clothing.

**On damaged skin or wounds requiring a secondary dressing:**

Apply a thin layer of Stratamed and then cover with the secondary dressing.

Drying is not necessary.

Stratamed should be reapplied when changing the dressing or checking wound progress, or as advised by your physician.

### Recommended duration of treatment

Stratamed is recommended for immediate treatment post procedure and should be continued until the skin has fully recovered.

For resurfacing and ablative procedures, continued use is recommended for a minimum treatment of 30 days (24 hours a day/7 days a week) or until no further improvement is seen.

### How much Stratamed do I need?

Stratamed gel is a unique formulation that requires substantially less product per application than typical moisturizing creams or barrier ointments.

- **Stratamed 0.17 oz (5g)** is enough to treat a 1–2 inch (2–4 cm) linear incision twice per day for over 90 days.
- **Stratamed 0.35 oz (10g)** is enough to treat a 2–3 inch (5–8 cm) linear incision twice per day for over 90 days.

- **Stratamed 0.7 oz (20g)** is enough to treat a 4–5 inch (10–12 cm) linear incision twice per day for over 90 days or an area of 2.5 × 5 inch (6 × 12 cm) twice per day for over 60 days.
- **Stratamed 1.75 oz (50g)** is enough to treat an area of 5 × 6 inch, (12 × 15 cm) twice per day for over 60 days.

### Caution

- For external use only.
- Stratamed should not be placed in contact with the eyes.
- Stratamed should not be applied over topical medications unless advised by your physician.
- Stratamed may stain clothing if not completely dry.
- Should your wound show signs of infection or fail to heal, consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for wet, highly exudative, tunnelling wounds or 3<sup>rd</sup> degree burns.
- Keep out of the reach of children.
- Store below 77 °F (25 °C).
- Do not use after the expiration (EXP) date printed on the tube.

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Stratapharma AG