[EN] Consumer information

Stratamed is a flexible, full contact, non-resorbable, occlusive wound dressing for damaged or compromised skin.

Stratamed is a self-drying, non-sticky, transparent gel formulation that lightly bonds to the most superficial damaged skin layer. When used as directed, Stratamed dries to form a protective wound covering that is gas permeable and waterproof (which hydrates and protects compromised skin areas and superficial wounds).

Stratamed can be applied to fresh incisions and excisions, open wounds and compromised skin surfaces including those from post laser treatments, chemical peels, dermal abrasion, 1st and 2nd degree burns and radiation therapy. Stratamed gel is bacteriostatic and inert.

For open wounds and compromised skin:

Stratamed dries to form a thin, flexible wound dressing which hydrates and protects affected areas. Stratamed promotes a moist wound healing environment for superficial wounds and burns.

Moist wound environments promote faster re-epithelialization[†], relieve itching, burning and discomfort, reduce redness and swelling associated with insulted skin.

Stratamed does not adhere strongly to the wound bed or the newly formed granulating tissue.

Stratamed can be used with or without a secondary protective dressing.

For abnormal scar management:

Stratamed softens and flattens raised scars and allows scars to normalize the collagen synthesis process, producing a normal mature scar.

Stratamed is used to relieve itching and discomfort and to reduce redness and discoloration associated with damaged skin and scars.

Stratamed can be easily applied to all areas of the skin, including exposed areas like the face and neck as well as joints and hairy areas without the need for shaving.

Stratamed was developed for the use on all types of superficial wounds, burns, skin tears and abrasions including those resulting from:

- General and cosmetic surgery
- Trauma
- Burns (1st and 2nd degree)
- Radiation therapy
- Bites
- Grazes
- Laser therapy, including tattoo removal
- Chemical peels
- Dermal abrasion
- Cryotherapy
- Acne and other diseases of the skin

Intended use

Stratamed is an occlusive wound dressing intended to cover a wound, to provide and support a moist wound environment, and to allow the exchange of gases such as oxygen and water vapor through the device.

Because Stratamed is a gel, it is easily applied to large exposed surface areas and contoured skin, like the face, neck and chest.

Stratamed is used for:

- The protection and hydration of superficial wounds, burns and compromised skin.
- For the management of all types of scars.

Stratamed may also be used in conjunction with invasive scar management options to improve overall results, including surgical excision and resurfacing, intralésional steroid injections, laser treatments and pressure garments.

Stratamed contains no steroids, alcohol, parabens or fragrances and is transparent and odorless.

Stratamed is suitable for children and people with sensitive skin. Stratamed may be used on infants, during pregnancy and while breastfeeding. Stratamed is intended for single patient use.

Directions for use

On first use of 0.17 oz (5g), 0.35 oz (10g) and 0.7 oz (20g) tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 1.75 oz (50g) tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the affected skin or wound is clean.

Gently pat dry as much excess exudate or wound fluid from the area as possible prior to gel application.

For best results Stratamed should be maintained in continuous contact with the skin (24 hours a day/7 days a week).

On damaged or broken skin not requiring a secondary dressing:

Apply a very thin layer of Stratamed to the affected area and allow the gel to dry.

When applied correctly to the affected areas, Stratamed should be dry in 5–6 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

Stratamed should be applied once daily, or twice daily to exposed areas or as advised by your physician.

Once dry, Stratamed may be covered by sunscreen, cosmetics, pressure garments or casts. If Stratamed has been removed by washing, it should be reapplied.

On damaged skin or wounds requiring a secondary dressing:

Apply a thin layer of Stratamed then cover with the secondary dressing. Drying is not necessary.

Stratamed should be reapplied when changing the dressing or checking the wound progress, or as advised by your physician.

Recommended duration of treatment

Stratamed is recommended for immediate treatment post procedure and should be continued until the skin has fully recovered.

For resurfacing and ablative procedures, continued use is recommended for a minimum treatment of 30 days (24 hours a day/7 days a week) or until no further improvement is seen.

For abnormal scar management, Stratamed is recommended for a minimum treatment of 60–90 days (24 hours a day/7 days a week). Larger scars may take longer. Continued use is recommended until no further improvement is seen.

How much Stratamed do I need?

Stratamed gel is an advanced formulation that requires substantially less product per application than typical creams or gels.

- **Stratamed 0.17 oz (5g)** is enough to treat a 1–2 inch (2–4 cm) linear incision twice per day for over 90 days.
- **Stratamed 0.35 oz (10g)** is enough to treat a 2–3 inch (5–8 cm) linear incision twice per day for over 90 days.
- **Stratamed 0.7 oz (20g)** is enough to treat a 4–5 inch (10–12 cm) linear incision twice per day for over 90 days or an area of 2.5×5 inch (6×12 cm) twice per day for over 60 days.
- **Stratamed 1.75 oz (50g)** is enough to treat an area of 5×6 inch, (12×15 cm) twice per day for over 60 days.

Caution

- For external use only.
- Stratamed should not be placed in contact with the eyes.
- Stratamed should not be applied over other skin treatments without the advice of your physician.
- Stratamed may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- Should your wound show signs of infection or fail to heal after 30 days of initial product application of initial product application consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for highly exudative wounds, tunneling wounds or 3rd degree burns.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Do not use if the tube is damaged.
- Stratamed does not require special disposal methods. Please follow your local disposal regulations.

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Stratapharma AG